DentaMile Desk MC-5

User Manual

Inhalt / Contents

User Manual – English	1
Benutzeranleitung – Deutsch	4
Manual do usuário – Português	7
用户手册 – 中文版	10
Instrukcja obsługi – Polski	12
Gebruikshandleiding – Nederlands	15

User Manual – English

Product Information

Instructions for Use

Please read through the following **Instructions** before you start using your printer.

Ensure there's no residuals inside the resin tank and the glass is well setup before printing.

Avoid print at center point with a single object or small parts repeatedly, it may damage the resin tank.

Regulatory Statements

Safety Certification

Safety certification is in progress.

Safety Precautions

Please read through the following Safety Precautions before you start using your printer.

- Use only the power adapter and accessories provided by the manufacturer.
- Before plugging the power adapter into an electrical outlet, verify the local power rating conforms with the specified voltage and frequency.
- This printer needs a grounded, 1-phase electrical power source.
- Make sure power and communication cables are well managed to prevent from tripping.

- Only use resins supplied by DMG manufacturer. Use of other resins may cause printing job failure.
- Do not expose the resin tank to direct light source because the resin will solidify under UV light.
- Chemical substances like resin may cause skin irritation, please read the guidelines in Material Safety Data Sheet (MSDS) of the resin supplied by manufacturer. It is recommended to wear gloves when necessary.
- When moving the printer for a long distance, be sure to withdraw the remaining resin in reaction tank and clean the tank in advance to prevent from resin overflow.
- Keep door closed during printing or curing to prevent UV leakage.
- UV light may cause injury of eyes, do not stare into the printer directly when it is in operation. It is recommended to wear UV protective glasses during operating the printer.
- Place the device on a level surface and always keep the printer in an upright position.
- Do not store the printer in extreme hot and cold temperatures.
- Do not disassemble or repair the device yourself. Doing so may cause device malfunction or danger.
- **Warning:** Picker is hazardous moving parts in *DentaMile Desk MC-5* Printer. Keep hands away from picker's motion path.
- Emergency shut down method: Press Power Button and hold 3 seconds to shut down.

This product is designed for specific application and needs to be installed by a qualified personal who has 3D printer and related rule knowledge. A qualified person understands the requirements and risks involved with installing electrical equipment in accordance with national codes.

General Information

Product Name	<i>DentaMile Desk MC-5</i>
Power Input	100–240V AC, 50Hz/60Hz
Adapter	DC 19.5V/6.67A
Wavelength	385 nm
Printer Dimension W x D x H	40 x 30 x 47 mm
Package Size and Weight	62 x 62 x 75 cm / 22 kg (Including pallet)
Operating Temperature	10° to 30°
Humidity (RH)	40% to 60%. Our printer should keep in dry conditions
Storage of Printing Material	Store the materials at ambient temperature from 15° to 28° and do not exposure to the light, otherwise resin may solidify
Liability: We will not hold the responsibility for damages of the machine or injuries on the operator in case user do not follow the user guide instruction. Repair can only be carried out by authorized person of DMG.	

Package Contents

There is a printer and an accessory box inside the package. As soon as you receive our products, please verify its content and check if there's any exterior damage due to transportation.

Setting up your 3D printer – Hardware

Location

Ensure your *DentaMile Desk MC-5* printer is placed on a flat table top with access to mains electricity. Avoid sunlight direct as this will degrade material left in the machine.


Warning! Dust pollution will cause printing failure. Do not place your printer in dusty environment, such as zirconia milling workplace.

Warning! Printer net weighs at least 20 kilograms. Please be careful and use proper lifting techniques to avoid injury.

Note: Do not discard your carton. Put it in a safe place should you need to ship your printer for any reason.

Product Overview

Front View
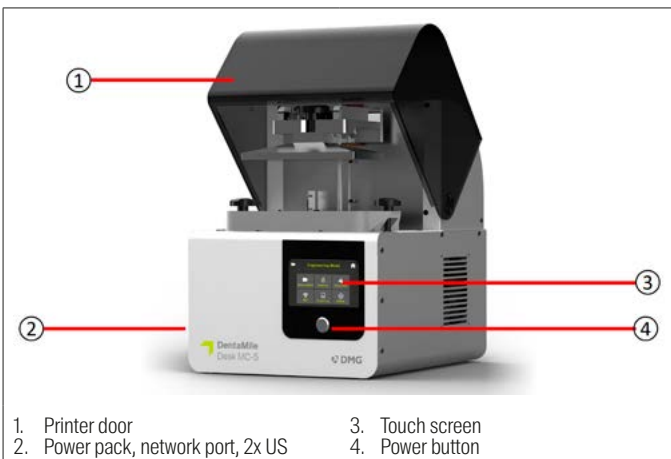


Connecting to the Power adapter and Cable (or plug adapter if needed)

1. Connect the power cord and adapter to your device, and then plug it into an electrical outlet
2. Connect your printer with laptop

Basic	Connect printer and laptop by RJ-45 cable	Initiating time 1 minute
LAN	Connect both printer and laptop to local area network	Initiating time a few seconds
IP sharer	Connect both printer and laptop to IP sharer	Initiating time a few seconds

Inside View

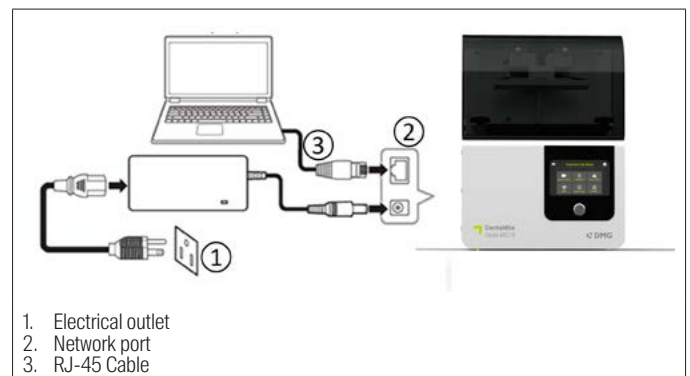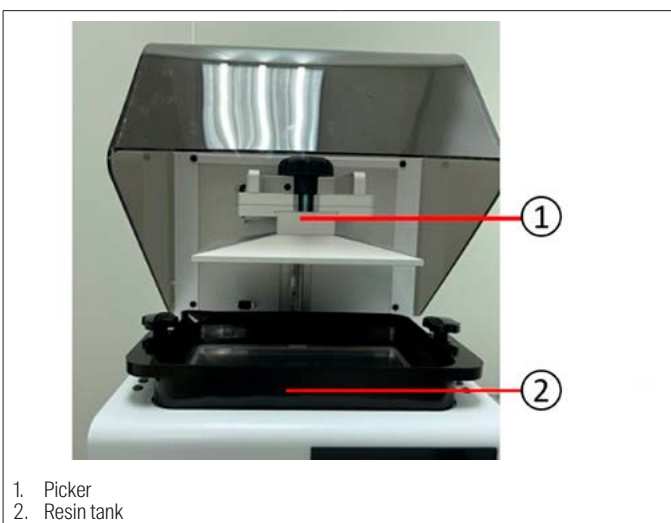Power Button Control

Power On

Press Power Button → Panel shows Starting up → Panel shows Menu

Power Off

Press Power Button → Confirm message → Confirm to shut down

Emergency shut down



Press Power Button and hold 3 seconds to shut down

Power Button Signal

Blue		Solid	Standby mode
		Blinking	Starting up or printing in progress
Red		Solid	Error

Install Picker

1. Carry the picker and install
2. Screw and lock the picker onto the machine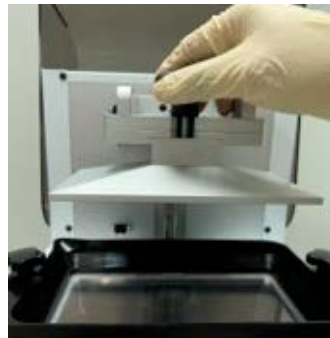
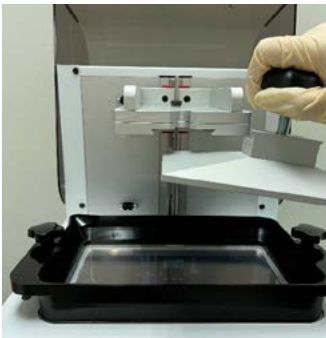


Remove the Picker




Unscrew the picker → Take out the picker

Fill resin tank

1. Unscrew the Resin tank from two point
2. Pick up the resin tank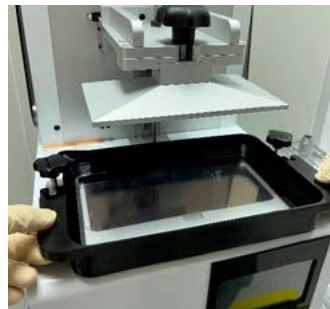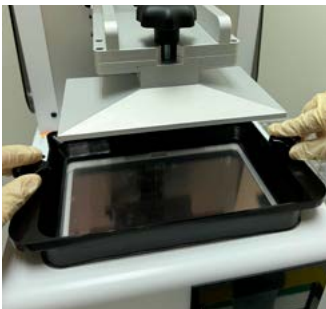

3. Pour Resin

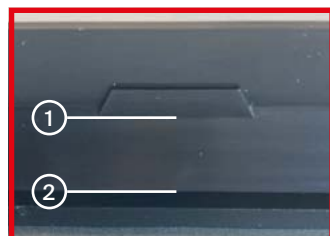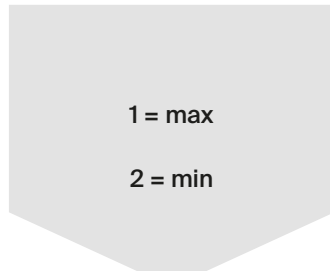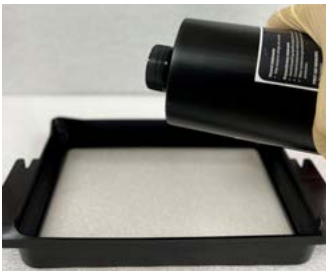Operate your 3D printer – Software

Please refer to DentaMile CAM MC User Manual.

After Print

Remove Model

Remove the Picker and printed model → Use air gun and ethanol to clean the printed model → Post curing (Additional post curing unit is required) → Carefully remove the printed model from the picker. → Use putty knife to help separate the printed model and picker

WARNING! Do not expose the resin tank to direct light source because the resin will solidify under UV light. Use tray to cover the resin tank to prevent solidify.

Clean Picker

Use putty knife or similar object to scrape away the excess printed model on the picker. If any greasy residue persists, use a soft cloth dipped in alcohol to rub the surface until the grease is removed.

WARNING! Do not use chisel to scrape off the printed parts on the picker. The rugged surface of picker will cause the resin tank & UV glass be damaged.

Clean Resin Tank

Pour resin out of tank → Pour ethanol into tank and soak in a while → Use tissue or lens tissue softly wipe the bottom of tank → Finished cleaning. Or let the projector help cleaning Tank, by engaging the clean out function on your printer.

Notifications

- Do not shut down the power during curing
- Do not interrupt the firmware upgrade process. Any interruptions during the firmware upgrade process may cause your printer to malfunction
- Update the firmware frequently for better user experience
- Before printing, please check the surface of picker and tank is clean, no residual, otherwise it may affect printing quality
- Regularly inspect the resin tank, and replace it with a new resin tank once tension status of Teflon film gets loose.

Specifications

Building size (mm)	130 x 73,1 x 90
XY Resolution	68 µm
Z-Layer Resolution	5 µm to 500 µm
Wavelength (LED)	385 nm
I/O (Connection)	Ethernet / USB
User Interface	5" Touch Panel
* All specification and design subject to change without notice	

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Young Optics Co., Ltd.